



iCassette™ Drug Screen


Corporate Drug Testing Made Simple.

- CLIA Waived Panels Available
- Runs on 3 mL of Specimen
- Laboratory Accurate
- 1-10 Drug Configurations Available
- Results in Minutes
- Simple Procedure

Innovative Drug Screening Solutions
Quality • Choices • Support

Instant Technologies, Inc.



an  inverness medical company

Detect the presence of drugs in minutes.
It's drug testing efficiency you can count on.




iCassette™ 5 Panel
shown above.


Drug Name:	Abbreviation:	Cutoff Levels:
Amphetamine.....	AMP.....	1000ng/ml
Barbiturates.....	BAR.....	300ng/ml
Benzodiazepines.....	BZO.....	300ng/ml
Cocaine.....	COC.....	300ng/ml
Ecstasy.....	MDMA.....	500ng/ml
Marijuana.....	THC.....	50ng/ml
Methadone.....	MTD.....	300ng/ml
Methamphetamine.....	mAMP.....	1000ng/ml
Morphine.....	MOP.....	300ng/ml
Opiates.....	OPI.....	2000ng/ml
Oxycodone.....	OXY.....	100ng/ml
Phencyclidine.....	PCP.....	25ng/ml
Propoxyphene.....	PPX.....	300ng/ml
Tricyclic Antidepressants.....	TCA.....	1000ng/ml

Instructions

- 1** Remove the test device from the sealed pouch.

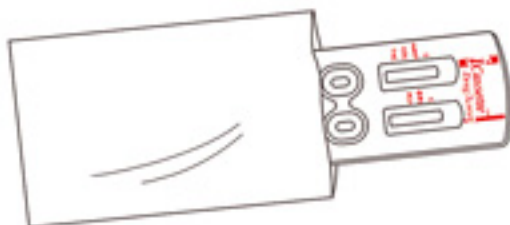

- 2** Hold the dropper vertically and transfer three (3) full drops of urine to the specimen well of the test device.

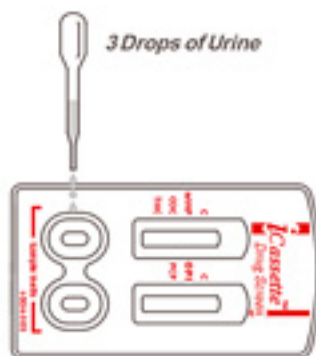

- 3** Read the results at five (5) minutes. Results stable up to four (4) hours.

